

INSTRUCTIONS FOR USE

UNIPATT RESIN code 1911929

1. Application

Building up patterns of outer copings with a brush.

Properties:

Colour: red
100 g powder
100 ml liquid

- Burns out completely
- Minimal shrinkage

2. Directions for use

Processing:

Dispense the required amounts of liquid and powder into separate mixing vessels.

- Dip the tip of the brush into the liquid and then pick up a small amount of powder.
- The resin swells on the brush tip and can be placed and shaped exactly on the coping.
- Repeat this procedure several times until the outer coping has been built up.
- Carefully raise the pattern from the inner coping and check the marginal fit as well as the inside of the pattern.
- Replace the pattern on the inner coping and use a cutter to trim it to uniform thickness.

Please note:

- If this material is being used on a plaster model, a separating agent should be applied.
- Clean the brush with the Unipatt Liquid after use or, if necessary, during use.
- Do not pour any leftover liquid or powder back into the bottle.

CAUTION!

If the user is known to be allergic to one of the constituents of this material, it must not be used.

Storage:

Store at room temperature (21°C) and do not expose to direct sunlight.

Danger:

Do not allow unpolymerized material to contact your skin and do not inhale monomer fumes.

Open packaging:

Do not use after the expiry date.

Important

This information is based on our present knowledge and thus are for guidance only. However, this shall not constitute a guarantee for any specific product features and shall not establish a legally valid contractual relationship. The consumer is solely responsible for product appliance, if it should differ from the stated range. As it is possible that the product is used in multi-purpose use far beyond our knowledge, we do not assume liability and responsibility for other special applications than those mentioned in our MSDS or instructions for use.

Despite of the terms of this MSDS, consumers have to assure themselves that the characterized product is suited for their individual intended purpose and that the safety appliance and the advices concerning environmental protection apply to their individual purpose and in their situation.